RISCHIO BIOLOGICO BIOLOGICAL RISK

PROGETTO ALLESTIMENTO **AREE EXTRACLINICHE**
EXTRACLINIC AREAS EQUIPMENT PROJECT

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Quality and reliability

Tecno-Gaz S.p.A. operates in a number of different fields with a strong tendency towards, and specialization in, the world of dentistry, focusing on the production of equipment and devices to improve the operational practices of professionals in the field through innovative solutions and products. **For more than 30 years** we have been setting ourselves the goal of finding more and more appropriate solutions to the demands of doctors and those who work with them, as well as creating products that guarantee safety and performance and can contribute towards improving **professional qualities**.

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Link immediati per Smart-Phone • Smart-Phone direct links

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Sales office: cme@tecnogaz.com • 05218380

Assistenza/ Services: service@tecnogaz.com

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Why the sterilization process

When evaluating risks within the medical environment, biological risk is considered the main factor to be analysed and brought under control. The environment and practices of medicine represent a risk for the transmission of pathogens, particularly those that are transmitted parenterally, if rigorous prevention methods to protect patients and operators are not applied. Within the medical environment, the transmission of agents of contamination can happen through:

- airborne pathways
- water contaminated by bio-film and back-flow of revolving instruments
- contaminated surgical instruments

The concept of prevention must be considered a very real cultural and structural project to be constructed with skill and care. **With Tecno-Gaz you can have the most complete, rational and safe plan for instrument sterilization;** no-one can count on solutions like ours, researched to make every activity safe and to guarantee maximum results. **Tecno-Gaz S.p.A.** can offer you solutions for fighting airborne infections and for the management and treatment of water, the latter being an issue of absolute importance for every medical facility.

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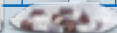
Unika
pagina / page 12

MultiSteril
pagina / page 9

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Sterilization test
and archiving system

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Tecno-Gaz project

Tecno-Gaz is the only company than can offer you a complete plan for managing every aspect linked to prevention within your facility.

Biological Risk

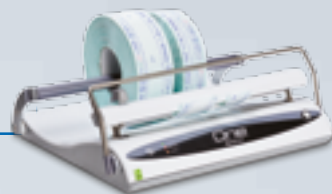
How to prevent infections in the field of dentistry

Unique Advantages

- making traditional sterilization more effective
- saving you time and money
- providing a maximum legal guarantee

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Frame and click! Use your mobile phone to get more information about this issue.

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Video Operating Protocols

Tecno-Gaz has created a teaching kit for all dental assistants, with regard to sterilization. The kit includes written procedures and a DVD with filmed procedures (procedures-control tests-useful recommendations).

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EN

The ultrasonic tub is designed to **wash instruments used in the medical, dental, cosmetic, veterinary fields of application and all those areas which require an ultrasonic washing**. The cleaning device involving ultrasound (via cavitation) allows you to thoroughly clean places otherwise difficult to reach by manual cleaning, without any risk resulting from unsafe handling of contaminated instruments.

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1 Trasduttori / 1 Transducer **Alto potere pulente**
High cleaning power

Sweep Mode **Pulizia efficace / High cleaning power**

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EN

The ultrasonic tub is designed to wash instruments used in the medical, dental, cosmetic, veterinary fields of application and all those areas which require an ultrasonic washing. The cleaning device involving ultrasound (via cavitation) allows you to thoroughly clean places otherwise difficult to reach by manual cleaning, without any risk resulting from unsafe handling of contaminated instruments.

Heating • FREE

The device proceeds to the heating phase of the detergent. It stops automatically when the preset temperature is reached. Always consider that during the ultrasonic cleaning the temperature of the detergent tends to rise independently.

Degassing (DEGAS) • FREE e BASIC

The device starts activating the ultrasonic phases intermittently. This process reduces the gas dissolved in the liquid, thus improving the efficiency of the subsequent washing.

Ultrasonic cleaning • FREE e BASIC

Washing phase through ultrasonic cavitation.

SWEEP • FREE e BASIC

Ultrasounds are modulated in frequency, thus reducing the negative effects of standing waves, increasing the distribution of ultrasonic energy within the washing liquid. The cleaning is more efficient (**improved cavitation**).

Discharge • FREE

- In the version without discharge (**BASIC 9**), the tub is to be emptied manually.
- In the optional self-discharge version (**FREE 9**), the tub is emptied by opening the appropriate valve.

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Support for 2 surgical trays (article code refers to support only)

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Alto potere pulente
High cleaning power

Sweep Mode

Pulizia efficace / High cleaning power

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EN

Artica vessels have been expressly studied to assure simple and controlled disinfection. All vessels are equipped with integrated timer support, total lid for easy liquid substitution and partial lid for the insertion of the instruments, avoiding the leakage of odours. Artica vessels are all supplied with plastic tray, removable on demand. **Artica line is composed by two models:**

- Artica 1 conceived for average loads, particularly suitable for surgeries.
- Artica 2 conceived for large loads, perfect for hospitals or big centres.

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EN

Clean is a new device created for instrument cleaning. The aim of this device is to help operators during the instrument cleansing phase, avoiding prolonged handling, thus making this phase simpler, more efficient and safer. **CLEAN is a simple device that includes the following:**

- a compartment for storing and placing the instruments to be treated.
- a central compartment complete with a coiled bristle to clean the instruments.

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EN

Tecno-Gaz S.p.A. has been able to create a new and revolutionary device that can automatically carry out all the major preparation phases for the sterilization cycle, such as disinfection, cleansing, rinsing, drying, without human intervention, without any professional risk and with a reduction in the overall time taken. A device that is unique in the world and indispensable for every facility that carries out sterilization.

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The aspects that Multisteril can help you manage correctly

Assistants' expenditure of time

Assistants spend a lot of time for the preparation phase to tools sterilization, because of multiple and complex manual procedures. **Multisteril does not require human presence.**

Risk factor

The manipulation of tools and potential infectious materials is a very important factor and it must be kept under strict control, both for operators' protection and the responsibility of the owner of the studio. **Multisteril cancels the risk.**

Management protocol

There is the sterilization if the procedure of preparation of materials is done properly, if not, the whole process is completely nullified. **Multisteril performs correctly all the cycles.**

Certified disinfection

Certificate issued by University of Dresden attesting cleaning capacity of Multisteril in combination with Multisteril CD cleaner.

Spaces

Space is another important problem, in fact, sterilization rooms often have small size and rationalization of space assumes a considerable value. **Multisteril can do all the process filling 40 cm.**

Overall costs

Cost is an important component which must be brought under control. **With Multisteril every cycle costs about 2 Euros.**

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Stand-by

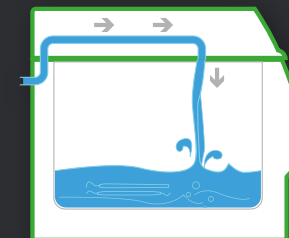
To improve cleansing, an initial standby phase has been created, which allows the tub to be filled with water and liquid detergent/disinfectant, so that the instruments sit better in the solution, **ensuring perfect cleansing, even when using bi-component cements.**

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Loading the water

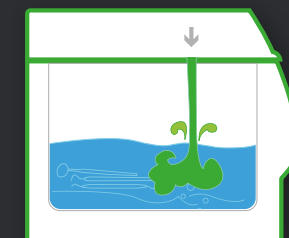
The water enters the tank **thanks to the direct connection to the mains water supply.**

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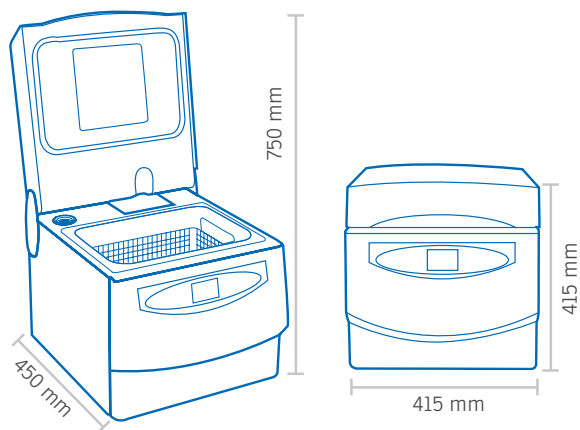
Disinfectant/detergent entry and liquid activation

A special peristaltic pump present inside the device automatically **introduces the percentage of disinfectant/detergent previously outlined in the parameters of the cycle.**

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Liquid concentrate
for the non protein-
fixing cleaning and
disinfection

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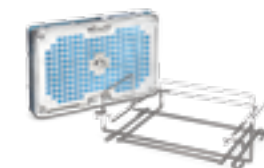
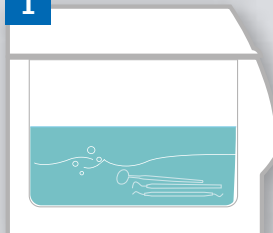
Support for 2 surgical
trays (article code refers to
support only)

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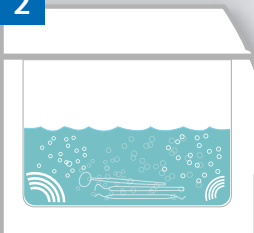
Heating

The device moves on to heating and to a **first static decontamination of the instruments**.

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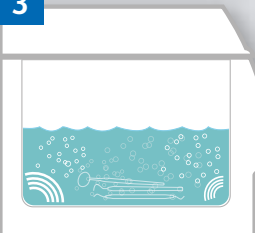
Degasification

This process eliminates the gases dissolved in the water avoiding interference with the ultrasounds and improving efficiency.

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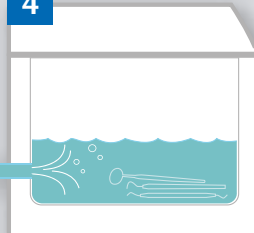
Ultrasonic wash

At the end of the degasification phase the ultrasonic cleansing begins, setting off the cavitation, which allows for the removal of chemical and organic residue present on the materials immersed in the tank, **destroying the bacterial load by more than 95%. The length of this procedure can be programmed by the operator for between 5 and 30 minutes.**

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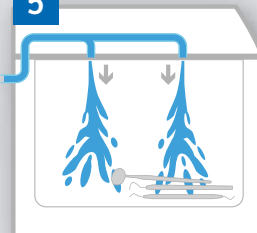
Waste

At the end of the cleansing phase, **MultiSteril automatically empties all the liquids present in the tank**. Once the tank is completely empty the following rinse phase begins.

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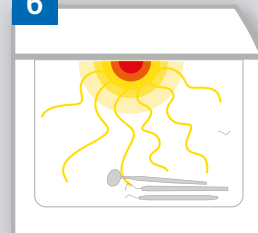
Rinse

To remove all chemical residue from the instruments, **the rinse phase begins, taking place through the pressurised intake of water through the special nozzles in the lid**. The water taken in is then automatically removed through the mesh. At the end the drying phase is launched.

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Drying

To guarantee a proper drying of the materials **the machine generates a flow of hot air, produced by a fan and a resistor located on the lid of the device**, which lightly touches the objects and takes away the dampness in the form of gas.

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EN

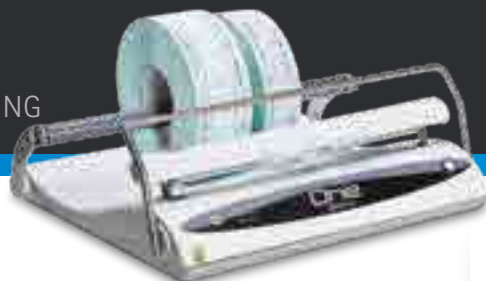
Demonstrating yet again its great attention to growing operating needs, Tecno-Gaz has produced a unique and extremely rational mechanical device for drying the materials to be sterilized. This device allows the obligatory drying to be performed and to avoid any handling of instruments especially sharp ones.

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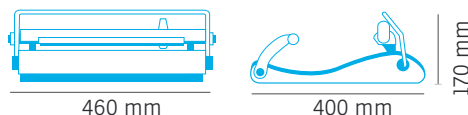
EN

ONE is a new generation sealer that has been created with the aim of reducing envelope packaging time, monitoring the quality of the welding, and is able to seal all types of paper for sterilization, including folded paper. **One assures:**

- Sealing band of 12 mm.
- Sealing bar 32 cm.
- Great easiness of paper introduction.
- Active electronic control over the couple of resistors.
- Visibility of the area of cutting.
- Signal of possible bad sealing.
- Introducible in any sterilizing piece of furniture.

Dati tecnici • Technical data	One
LxAxP/ WxHxD	(460x170x400) mm
Peso/ Weight	8 Kg
Assorbimento max./ Max absorption	320 W
Tensione/ Power supply	230 V~ - 50/60 Hz
Larghezza saldatura/ Seal width	12 mm
Lunghezza saldatura/ Seal length	320 mm

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EN

Tecno Seal is a roller sealing machine with photocell automatic feed. Tecno Seal is equipped with built-in printer (1TAUA0002) to print on pouches packaging and expiry dates as well as symbols during the sealing phase. This solution allows to have constant control on the state of sterility of instruments. On one side of the machine there is an electronic control that allows the management of the parameters of sealing.

- Automatic photocell feed system (10 mt/min).
- Complete with integrated printer (Art. 1TAUA0002)
- Automatic printing of seal/expiry date.
- Electronic card for control/management of parameters of use.

Dati tecnici • Technical data	TecnoSeal
LxAxP/ WxHxD	(555x250x145) mm
Peso/ Weight	14 Kg
Assorbimento max./ Max absorption	500 W
Tensione/ Power supply	230 V~ - 50/60 Hz
Larghezza saldatura/ Seal width	12 mm
Lunghezza saldatura/ Seal length	Continua illimitata/ Continue unlimited

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EN

Ekstra Plus is a dental device for the **automatic maintenance of turbines, handpieces**, of all brands. The system is able to perform the **disinfection, cleaning and lubrication** selectively on **turbines, and handpieces** simultaneously in an **automatic way**. **Ekstra Plus** requires **electrical and pneumatic connections and air**, while the **liquid disinfectant and oil tanks are integrated** into the device. Very useful device in a dental practice. **For those**

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who want to minimize occupational risks and to maximize the operation and prestige of the dental practice.

All disinfection, cleaning, drying and oiling phases occur automatically, thanks to a process managed by a powerful microprocessor.

Dati tecnici • Technical data

	Ekstra Plus
LxAxP/ WxHxD	470X510X370 mm
Peso/ Weight	24 kg
Alimentazione elettrica/ Electric power supply	230 V (±10%)
Alimentazione pneumatica / Pneumatic air supply	5bar 100 l/minuto

Fasi del processo | Phases of the process

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Ekstra Plus is equipped with a SOFTWARE able to **control the phases of the process and store all the characteristics of the treatment by sending them to an SD card.**

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EN

Tecno-Gaz spa has introduced a new series of sterilization bags and reels called T paper.

These devices are designed for hospital, medical, dental, cosmetic, veterinary and podology uses and, in general, for all the personnel that carries out steam or gas sterilization processes.

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mm 55x200 mt.	Conf. 8 Pz./ Pack 8 Pcs.	Art. CS002Z01
mm 75x200 mt.	Conf. 8 Pz./ Pack 8 Pcs.	Art. CS003Z01
mm 100x200 mt.	Conf. 4 Pz./ Pack 4 Pcs.	Art. CS004Z01
mm 150x200 mt.	Conf. 4 Pz./Pack 4 Pcs.	Art. CS005Z01
mm 200x200 mt.	Conf. 2 Pz./Pack 2 Pcs.	Art. CS006Z01
mm 250x200 mt.	Conf. 2 Pz./Pack 2 Pcs.	Art. CS007Z01
mm 300x200 mt.	Conf. 2 Pz./Pack 2 Pcs.	Art. CS008Z01
mm 400x200 mt.	Conf. 1 Pz./Pack 1 Pc.	Art. CS015Z01

Buste autosigillanti	90x250 mm	Art. CS011Z01
(200 pz. per conf.)	140x260mm	Art. CS012Z01
Self-sealing bags	190x330mm	Art. CS013Z01
(200 pcs. pack.)	300x450mm	Art. CS014Z01

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EN

The tracking of the instruments is a basic procedure for the handling of sterilized materials. Tracking guarantees the recording of the "path" taken by each and every instrument, from preparation to sterilization, until it is used. This is made possible thanks to a data cross checking management system. Tracking is an activity that forms an integral part of the sterilization cycle procedure and should be implemented after the wrapping phase and before the autoclave is loaded. **This procedure is indispensable in order to guarantee maximum legal protection to the operators and to the health director.**

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CD contenente file moduli stampabili

Moduls specifically devised for recording sterilization cycles.
CD containing files with printable forms.

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* modello Andromeda Vacuum Plus Xp (camera 20 lt.)

* model Andromeda Vacuum Plus Xp (20 lt. chamber)

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EN

Autoclave is the key device within each medical structure, therefore it is fundamental choosing a well-known and referenced product which can rely on a widespread network of technical support. Onyx B is the new range of Tecno-Gaz autoclaves, a world leading Company for many years.

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P.I.D. System

P.I.D. means Proportional Integral Derivate and represents the apex of closed loop controls. The structure of this technologically advanced control is simple and flexible. P.I.D. ensures a constant control and monitoring of the whole operating system of the autoclave, acting automatically in case of anomalies or incorrect values in technical and electronic parameters. **This device behaves like a "real technician" with real-time diagnostic and intervention capability. This solution reduces anomalies and warnings, granting maximum reliability and minimising the machine inactivity times.**

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SD card

SD CARD system allows **constant machine updating** by a simple download on it and then by inserting it into the autoclave system. **An extraordinary option in order to have constantly efficient and updated device.**

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Double load... Same time

Using standard size autoclave about 4 kg instruments can be sterilized, while with Onyx B you can sterilize from 5 kg to 8 kg of instruments. **With 1 double load you are saving 50% of your time.**

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Cycles for all needs

Flexibility is fundamental to rationalise working and create operative efficiency. **Onyx B autoclaves offer a wide range of cycles all studied to help you every day.**

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Best management of instruments

Having big load autoclave helps you to rationalize instruments flow and increases your operative efficiency. Efficiency means profit, Onyx B autoclaves are advantageous for this too.

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Energy saving

Onyx B autoclaves have low energetic consumption as well as unbeatable consumption/ load ratio. **Economic saving and benefit for the environment.**

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Water net connection

The autoclaves Onyx B are autonomous as equipped with double tank of pure water and exhausted water, they are **prepared for the connection to the water supply through osmosis system mod. PURA Plus.**

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Big load

New regulations and new sterilization concept have developed sterilization practice in all medical and dental facilities along with wider range of instruments and spread of professional facilities which developed the need of autoclaves able to sterilize large loads. **Onyx project is born with the aim to satisfy these need adopting specific and new solutions enabling to manage easily also partial loads.**

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Proper trays for any type of load

Trays of different lengths enables you to insert instruments inside bulky envelopes, to place load more rationally enabling sterilization of **different loads following the correct guidelines for load placement.**

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FL System - Full Load System

To avoid manual loads with possibility of water pouring off, Onyx B autoclaves throughout their automatic clean water tank automatic system are using a level sensor control pump with automatic lock. **Frontal loading, no opening cover or removing system, enabling simple placement of the machine.**

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ID system

Authentication integrated system of the operator which is protected by password and permits at the end of the cycle to associate operator to sterilized load having visual convalidation of the operator himself. The same authentication is also reported by the procedure of load traceability.

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PHD System

Preheating system with stand-by for minimum temperature **maintenance and consequent overall sterilization cycles time reduction.**

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Inspectionable tanks

In case of insertion of unsuitable liquids, autoclave tanks are inspectionable and cleanable as per technical normatives regulations. **Onyx B autoclaves are supplied with pure and used water tanks.** Each tank is supplied with probes indicating levels.

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Overnight cycles

Automatic overnight cycles with anti-condensation system and automatic switch off.

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Reduced maintenance

Onyx B autoclaves allow you to make big loads thus reducing the number of cycles. This means **extending autoclave life over 50% reducing maintenance costs and amortizing the cost of the product over the time.**

CICLI OPERATIVI STUDIATI PER L'EFFICIENZA DEL PROCESSO OPERATIVE CYCLES CONCEIVED FOR PROCESS EFFICIENCY

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7 sterilisation cycles

Onyx B is born with the aim of rationalizing the work of sterilization operators, therefore the autoclaves are equipped with cycles which have been expressively conceived for improving time management and operator's needs within outpatient clinics.

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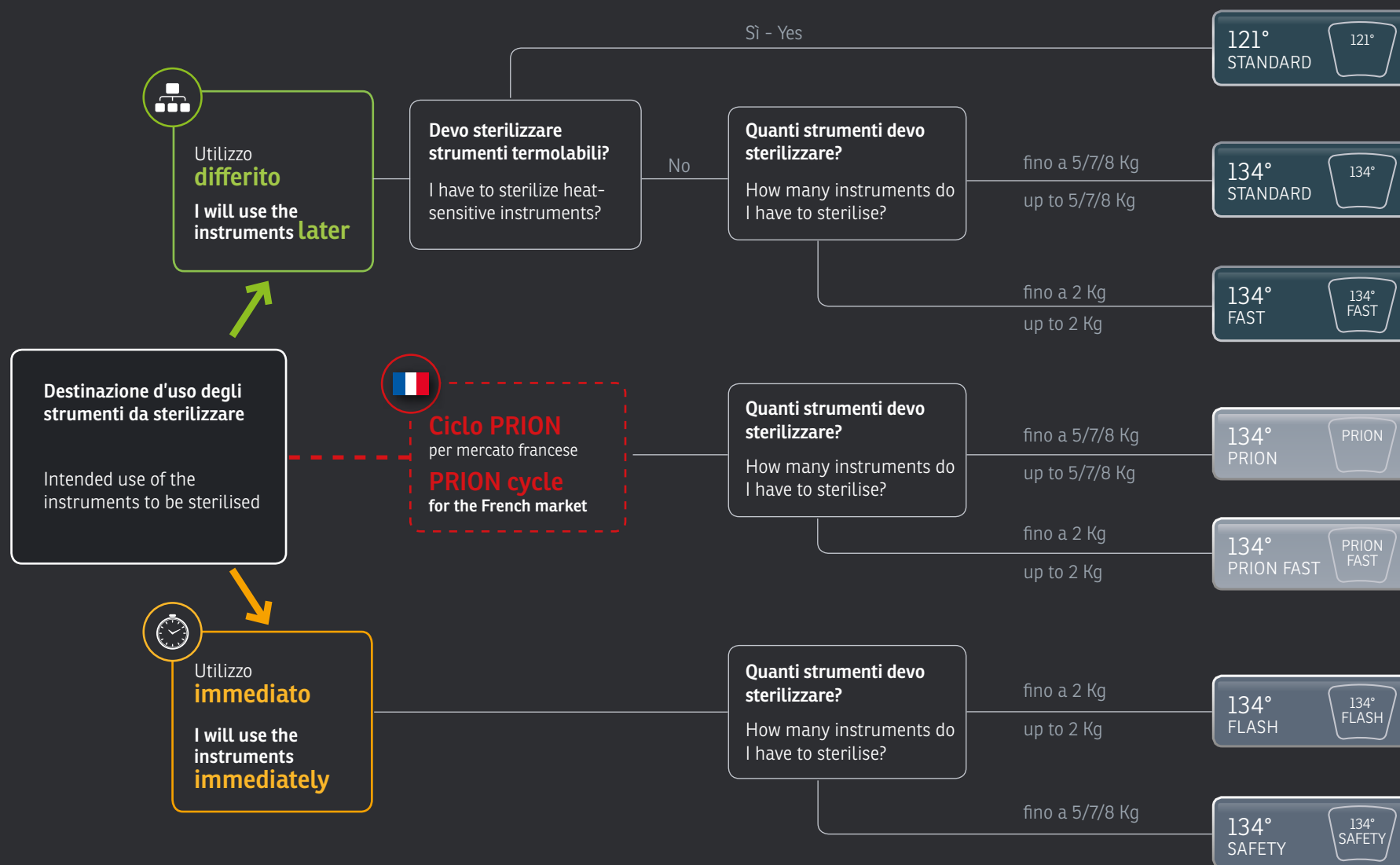
Completely automatic cycles

All the cycles are completely automatic, **all the phases are managed by the advanced card complete with powerful microprocessor**. Each and every phase (vacuum phase, preheating, exposure time, drying) is monitored, backed up and managed automatically. There is no human involvement in the carrying out of the cycles.

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Europa B Pro is an autoclave designed using new technology suitable for the sterilization of type B loads. Europa B Pro can count on sophisticated solutions, which make it the autoclave with the best quality/price ratio on the market.

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Electronic control

Europa B Pro is managed by a new electronic system, equipped with a latest generation micro-processor.

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Sterilization cycles

121°C-134°C suitable both for wrapped and unwrapped materials. All cycles are for type B loads.

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Cooling system

New cooling system with a double-bodied radiator.

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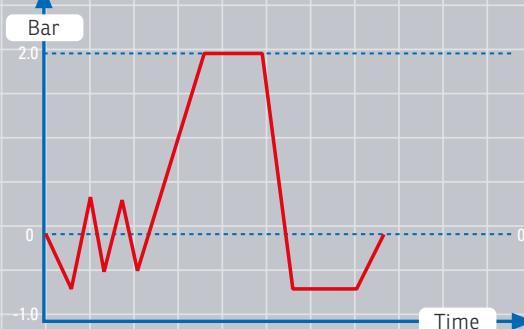
Technical characteristics

Pressure dynamics during a sterilization cycle with Tecno-Gaz Europa B Pro autoclaves.

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Safety on the door

We have applied three safety devices on the autoclave door:

- **The first device is a mechanic and electromagnetic one:** it triggers when the sterilization cycle begins and completely blocks the door opening. This device can be removed at the end of the cycle by pushing the device-unlocking button, but only if positive pressure is near zero.
- **The second device is an electrical one:** it triggers automatically when an attempt to force the door open is detected.
- **The third device is dynamic** and acts on the structure of the gasket.

Safety devices in place

- Certified safety valve.
- Ped on sterilization chambers.
- Insulation of the chamber with certified ceramic material.

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Barometric alignment

Automatic barometric alignment is provided from the first time the device is switched on, without any technical intervention.

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Overnight cycles

Automatic overnight cycles with anti-condensing system and automatic switch-off, with memory effect.

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Provision for mains water supply

The osmosis system mod. Pura Plus is an extremely compact piece of equipment for osmosis, created to be directly connected to the autoclave, guaranteeing a constant supply.

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SD Card

Collection and archiving of data relating to the cycles and the monitoring of the autoclave are carried out through an SD card. This system also allows for the software of the autoclave to be updated without any technical intervention, but simply by inserting a suitable SD CARD.

ACCESSORI • ACCESSORIES

ART. 276-S

Memory card

ART.N.SPLA349

Speciale portatray bivalente

Special dualpurpose tray carrier

ART. 2110S

TecnoPrint
Stampa etichette

TecnoPrint
Printing traceable labels.

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P.I.D. means Proportional Integral Derivate and represents the apex of closed loop controls. The structure of this technologically advanced control is simple and flexible. P.I.D. ensures a constant control and monitoring of the whole operating system of the autoclave, acting automatically in case of anomalies or incorrect values in technical and electronic parameters. This device behaves like a "real technician" with real-time diagnostic and intervention capability. **This solution reduces anomalies and warnings, granting maximum reliability and minimising the machine inactivity times (if any).**

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EN

Europa B Evo is an autoclave that has been developed jointly by the Tecno-Gaz research office and important external development facilities. Europa B Evo is indispensable for all medical and dental facilities and in all fields where safe sterilization of all materials is required: hollow, porous, solid, wrapped and unwrapped. Europa B Evo continues the success that Tecno-Gaz has achieved over the years, in fact, thousands of professionals use Tecno-Gaz autoclaves and are very satisfied.

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Built-in printer

Built-in printer for the registration of cycles carried out. The printer includes:

- date, time
- cycle number
- serial number of the device
- minute by minute cycle data
- outcome of the cycle

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**For the sterilization of
type S loads.**

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EN

Andromeda Vacuum XP is provided with a powerful vacuum pump which, at the beginning of the cycle, ejects the airpockets which are into the sterilization's chamber, and, at the end of the sterilization's phase, grants a very good drying. **A sophisticated electronic device grants a constant monitoring of pressure, temperature and time.** It can work also without the presence of the operator, as all the functions are automatic. Andromeda Vacuum XP has an electromagnetic block device on the door. The total automatism of each phase of the cycle prevents from any possible human error, also from inexperienced personnel. Stainless steel chamber, uniformly heated by a band heating element. Twin tank for clean and used water. **Andromeda Vacuum XP is provided with 5 cycles of sterilization: 121° C and 134° C for unwrapped and wrapped instruments, and a "flash" cycle for the fast sterilizations.** Andromeda Vacuum XP is supplied together with the printer built-in-into the drive board.

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**For the sterilization of
type N loads.**

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EN

Hydra Evo is steam autoclave for the sterilization.

- **2 sterilization cycles:** possibility of choosing two cycles at 134°C or 121°C.
- **Automatic phases** of preheating, sterilization cycle and drying.
- **Closure system with an operator safety pivot.** Air pockets automatic ejection system: thermodynamic system for the ejection of the "cold" air pockets.
- **Stainless steel chamber:** the sterilization chamber is made entirely of stainless steel and is heated by a band shaped heating element which guarantees uniformity of heat.
- **4 aluminium trays.**
- **Electronic control with digital display,** allowing the digital display of times and temperature.
- **Control of the sterilization process,** through microprocessor management with a constant measurement system.
- **Operation control, blocking the autoclave** in the event that there is a lack or an inadequate quantity of water in the chamber.

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EN

The Pura osmosis system is extremely compact and it's designed for connection directly to the autoclave, to guarantee a constant supply of pure water. The system can be installed in a number of positions to facilitate mounting and the optimum organisation of the available space. **Pura is fitted with a series of osmotic membranes and filters**, with an average duration of about 800 cycles, depending on the original hardness of the water. The cartridge should therefore need replacing about once a year, or after 800 cycles. **The overall cost of the filters is very reasonable, and the cost per litre is therefore almost negligible. When the Pura system is not producing water, it is not pressurized, and this avoids all kinds of problems caused by the accidental breakage of couplings and fitting.**

Why an autoclave osmosis system?

The osmosis system is recognised as the best method, ensuring treated water free from impurities and hard particles in order to protect the workings of the autoclave. The system allows for a direct connection with the autoclave, ensuring service and a reduction in costs relating to the purchase of distilled water.

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EN

Devices for supplying autoclaves

This is a simple, safe and rational system that ensures optimum quality distilled water, thanks to a boiling procedure with forced recondensation. **The use of Distiller will avoid the purchase and storage of cumbersome water barrels.** Distiller is supplied complete with accessories, filters and other equipment.

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EN

Biological control test for autoclaves

Our biological incubator is a product developed for the incubation of biological indicators. The incubator consists in a frame, an ABS cover, an aluminium heating unit with 3 slots for test tubes/biological indicators and an electronic card for automatic control of heating unit temperature.

What are these used for?

This can be used by **all those who own an autoclave**.

Why choose a Tecno-Gaz incubator?

Because they are **certified incubators**.

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3 phial biological incubator, with **spy operation and master switch**, connected to mains electricity, 220 V~.

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EN

15 phial biological incubator, with **spy operation and master switch**. Dual temperature selector 37°C for ethylene oxide, 57°C for steam, connection to mains electricity, 220 V~.

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What are these used for?

Biological test suitable for all types of **steam autoclaves**. It is used to assess the ability of the autoclave to destroy all micro-organisms including spores. **Requires the use of spores in phials and a biological incubator**.

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EN What are these used for?

Physical test, only suitable for autoclaves that are able to sterilize porous loads. **It is used to assess the ability of steam penetration in porous bodies. Requires the use of spores in phials and a biological incubator.**

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EN What are these used for?

Tests suitable for all types of autoclaves. They are used to assess the ability of colour change in the autoclaves through the parameters of the machine time temperature - pressure.

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EN What are these used for?

It is used to assess the **ability of steam penetration in hollow bodies.**

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EN Norm

- Useful numbers organization
- User manual
- Declaration of conformity
- Installation report
- Ordinary maintenance program
- Periodic maintenance program
- Maintenance book
- Technical interventions copy

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EN

Safeguard your professional image and your prestige, by protecting your health and that of you colleagues and patients. SterilAir PRO is the first air sterilization system. SterilAir PRO is an air sterilizing appliance which guarantees the elimination of all microorganisms including spores. **This device can run continuously also in the presence of people, ensuring maximum safeguarding of the operators.** SterilAir PRO is simple, silent and ergonomic.

Functioning: its functioning is based on a system of forced ventilation closed loop. The air drawn into the module first passes through a filter, mechanically positioned, within the inlet entrance. Here the coarser pollutants are blocked, preventing the fouling of the germicide lamps. Then the air is forced to pass in direct contact with mercury vapor tubes that, thanks to the emission of the UV-C, carry out the maximum germicidal action. The high reflectivity shielding action of the housing concentrates the reflections of the UV-C radiation. The air is subsequently expelled through the nozzle output, thus leading to an effective microbial killing.

The system: the great advantage consists in the absolute lack of danger for the operator since there is the minimum emission of UV-C radiations from the unit. The device can effectuate a continuous air disinfection treatment (120 m³/h). It's possible to carry out a demolition of microbiologic pollution of the air in every room, during every productive phase without any damage for operator's health. **After about three hours from installation 90% of demolition of bacterial charge in the air.**

	SterilAir Pro
Emissione esterna raggi UV-C UV-C rays external output	nessuna / none
Schermatura interna Internal shield	Alluminio UVC riflettente Reflected UVC aluminum
Pericolosità/ Pericolo	nessuna / none
Emissione di ozono/ Ozone output	nessuna / none
Colore/ Colour	RAL 9006
Installazione/ Installation	A parete e a stativo Wall mounted and stand mount

Note - Notes

Note - Notes

Handwriting practice lines consisting of 20 horizontal dotted lines on a white background.

Strada Cavalli, 4 • 43038 Sala Baganza (PR) Italy
Tel./Ph +39 0521 8380 • Fax +39 0521 833391 • info@tecnogaz.com

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All rights are reserved. Any variations on images and subjects can be done without notice. Tecno-Gaz S.p.A is not to be considered responsible for damages caused by lack or wrong information here mentioned. Product images are only for advertising purposes.