Extra info

1600°C

Product info

Dimensioni (LxAxP)
Dimension (LxHxP) **550 x 630 x 600 mm**
Dimensions (WxHxD)

Peso
Poids **115 Kg**
Weight

Tensione
Tension **230 V - 50/60 Hz**
Main voltage

Pressione aria
Pression d'air **6 bar**
Air pressure

Vuoto
Dépression **0,7 bar**
Vacuum

Temperatura
Température **0-1600 °C**
Temperature

Altezza cilindri
Hauteur cylindres **60 mm**
Cylinders' height

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En

ASM 30 CASTING MACHINE IS A NEW GENERATION, FULLY AUTOMATIC PRESSURE VACUUM CASTING MACHINE.

The entire casting process is made up of very simple steps: vacuum production, inverting, compression and cooling, with no human intervention required. Thanks to its special resistance, it can melt any type of alloy, precious, Palladian, base, semi-base as well as alloys containing titanium. ASM 30 is equipped with an SD card for recording and filing casting data. If requested, the machine can be equipped with an Argon gas device.

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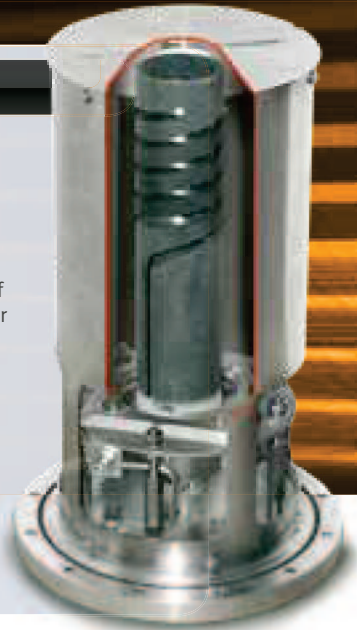
HEATING PART MORE AND MORE HIGH PERFORMANCE AND EFFICIENT

The heating part has undergone various updates over the years and has now reached an optimum level of technical efficiency and reliability. Our heating elements have always been extraordinarily successful at melting base metals, chrome cobalts, steel etc..., but we can now also guarantee long life, durability in use and resistance to thermal stress. **A lot of reasons to replace your casting machine and adopt new ASM30 automatic system.**

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CRUCIBLES

The crucible is an essential component for a die-cast casting machine, since it is the element that ensures correct irradiation and heating of the mass to melt. A non original pot can damage the heating element beyond repair or cause quality and casting problems.

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FURNITURE FOR CASTING MACHINE

The cabinet is an important accessory because it has been specially designed to display the inside of the pot. The cabinet is equipped with pot-holders, pump housing and accessories shelf.

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VACUUM PUMP

In compliance with EC directives, the casting machines are equipped only with Tecno-Gaz vacuum pump.

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ARGON DEVICE

The casting machines can be equipped with argon devices that make it possible to reduce oxidation during the casting process.

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LARGE INTUITIVE DISPLAY

The large display features graphic symbols, intuitive programming and synoptic visualisation for all the functions. The control panel only has a few multifunction buttons, which allow rapid programming and ease of use in operation.

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En

NEW SOFTWARE FOR TOTAL CONTROL OVER THE MACHINE

The electronic control part is the fruit of research aimed at creating a new system for the overall management of the machine, which would allow constant monitoring of the apparatus, plus simple and accurate programming.

- Intuitive commands and menu
- Multifunction buttons
- Clear information
- Large size display

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GAS, OX TAPS for simultaneous use of welding and fusion blowpipes.

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FUSION BLOWPIPE: UNIVERSAL handgrip T3 with lance for noble metals, gas/ox tubes with couplings. Optional lance for poor alloys.

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En

Universal Welder

is a micro-welder for dental technicians and goldsmiths. Equipped with wheels and with minimum overall dimensions, it can easily be positioned in limited environments. The two bottles (oxygen, L.P.G.) can be recharged, do not require periodic overhauling and are in compliance with laws currently in force. Simple and ergonomic, UNIVERSAL WELDER is equipped with a bottle-carrying trolley, rechargeable steel bottle for oxygen (5 LT.), rechargeable steel bottle for L.P.G. (3 LT.), pressure reducers with explosion-proof safety valve. Case containing UNIVERSAL handgrip, 4 welding tips with different diameters (0.3 mm, 0.75 mm, 1 mm, 2 mm), gas/oxygen tubes with couplings. Apparatus available also with fusion blowpipe (art. 4025/S).

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En

This new UNIGAS bunsen burner with thermocouple was designed to comply with the latest law regulations. It can be used with all kinds of gases and at any pressure thanks to a new universal nozzle. The thermocouple is composed by a valve block and a sensor unit. This thermocouple is a safety device which stops the gas flux in case the flame goes out, avoiding a possible gas saturation of the room.

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En

Skylab is an apparatus which encloses three sandblasters in one!

It comprises three independent boxes, each equipped with a control dashboard where switches for lighting, aspiration, air pressure reducer and pressure gauge are found. SKYLAB allows the simultaneous use of three types of sand with automatic reintroduction of sand into the work cycle. The sand jet is produced by depression rather than by compression, thus avoiding blockages, reducing the air consumption and eliminating humidity. Efficient twin action method of aspiration for removing fumes and dust: the first sucks silently during the working phase, the second allows the cleaning of the cabinets and the removal of the contaminated sand, which is held back in a special sack filter. SKYLAB can rotate on its axis through 240°. CE marking.

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En

Nybra is a sandblaster which uses two different kinds of sands.

Thanks to its twin section dashboard it is possible to control the lighting, the aspiration and twin working pressure. NYBRA does not have sands under pressure and the sand jet is achieved through depression, exploiting the Venturi method. This method significantly reduces the air consumption, prevents blockages and the collection of humidity in the sand. NYBRA is equipped with an efficient twin action method of aspiration for removing fumes and dust. The first sucks silently during the working phase, the second allows sand to be removed using the special lance supplied. NYBRA comes complete with a filter sack to hold back the sucked dust and an absolute filter to guarantee the total purification of the air discharged into the atmosphere. CE marking.

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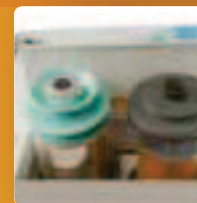
Nybra: sandblaster with 2 independent sand units

3

Filtri aria

Filtres à air

Air filters

Product info

Diametro Diamètre Diameter	415 mm
Altezza Hauteur Height	345 mm
Profondità Profondeur Depth	530 mm
Tensione Tension Main voltage	230V - 50/60 Hz
Consumo d'aria Consommation d'air Air consumption	200 NL/min.
Peso Poids Weight	29 Kg
Assorbimento Consommation électrique Power absorption	1 Kw